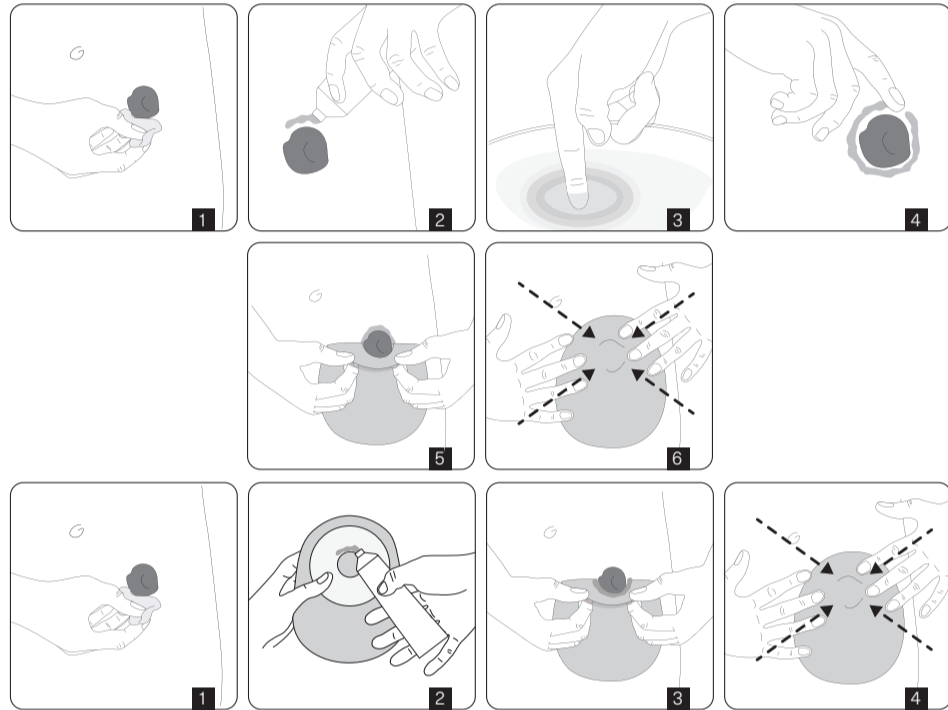
	Pročitajte upute za uporabu / Pročitajte uputstvo za upotrebu / Glejte navodila za uporabo / Consult instructions for use / Gebrauchsanweisung / Consultar las instrucciones de uso / Συμβουλευτείτε τις οδηγίες Χρήσης / Consultare le istruzioni per l'uso		Držati dalje od sunčeve svetlosti / Držati dalje od sunčeve svetlosti / Hranite stran od sončne svetlobe / Keep away from sunlight / Vor Sonnenlicht schützen / No exponer a la luz solar / Διατηρείτε μακριά από το ηλιακό φως / Tenere al riparo dalla luce solare
	Jedinstvena identifikacija proizvoda / Jedinstvena identifikacija proizvoda / Edinstveni identifikator pripomočka / Unique device identifier / Einmalige Produktkennung / Identificador único del producto / Μοναδικό αναγνωριστικό τεχνολογικού προϊόντος / Identificazione unica del dispositivo		Internetska stranica s informacijama za pacijente / Internetska stranica sa informacijama za pacijente / Spletna stran z informacijami za pacijenta / Patient information website / Website mit Informationen für Patienten / Sitio web de información para pacientes / Ιστοσελίδα πληροφοριών για τον ασθενή / Sito web con le informazioni per i pazienti
	Oznaka CE / CE oznaka / Oznaka CE / CE mark / CE-Kennzeichnung / Marca-do CE / Σήμανση CE / Marcatura CE		Broj serije / Broj serije / Številka serije / Batch code / Chargenbezeichnung / Código de lote / Κωδικός παρτίδας / Codice del lotto
	Kataloški broj / Kataloški broj / Kataloška številka / Catalogue number / Bestellnummer / Número de catálogo / Αριθμός καταλόγου / Numero di catalogo		Čuvati suhim / Čuvati na suvom / Hranite na suhem / Keep dry / Vor Nässe schützen / Mantener seco / Διατηρείτε στεγνό / Tenere al riparo dall'umidità
	Medicinski proizvod / Medicinski proizvod / Medicinski pripomoček / Medical device / Medizinprodukt / Producto sanitario / Ιατροτεχνολογικό προϊόν / Dispositivo medico		Datum proizvodnje / Datum proizvodnje / Datum proizvodnje / Date of manufacture / Herstellungsdatum / Fecha de fabricación / Ημερομηνία κατασκευής / Data di fabbricazione
	Rok uporabe / Rok upotrebe / Uporabno do / Use-by date / Verfallsdatum / Fecha de caducidad / Ημερομηνία λήξης / Data di scadenza		Proizvođač / Proizvođač / Proizvajalec / Manufacturer / Hersteller / Fabricante / Κατασκευαστής / Fabricante
	Distributer / Distributer / Distributer / Distributor / Verteiler / Distribuidor / Διανομέας / Distributore		

**A**

**B**

**HR**

Upute za uporabu

**OPIS: Lentell OstoPaste.** Pasta za stome. Pomagalo za stome

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## Instructions for use

**DESCRIPTION: Lentell OstoPaste.** Stoma barrier paste. Ostomy appliance.

**INTENDED PURPOSE:** Fills indentations or scarring in the untouched skin around the stoma of colostomy, ileostomy and urostomy patients. It creates an impermeable and equal surface between the stoma and the base plate to protect the skin from contact with the stoma output and to secure base plate adhesion. Be sure to strictly follow the instructions for use and to contact your stoma nurse or doctor if necessary.

**CAUTION:**

- Rely on the guidance of medical personnel while using the product. Our company does not bear any responsibility for indirect damage or losses caused by the use of product that does not comply with the instructions for use.
- Apply directly to the skin around the stoma. For use on intact skin only. Avoid delicate or sensitive areas of skin (such as eyes, mouth, ears, nose, vagina).
- Patch test a small area prior to use. Discontinue the use if irritation or allergies occur.
- Avoid contact of the tube opening with water or the stoma.
- If the paste changes colour or its continuity is disrupted, throw it away.
- Keep the product away from fire.
- If during the use of this device or as a result of its use a serious incident occurs, please report it to the manufacturer and to your national authority.
- Do not use the product after the expiration date.

**CONTRAINDICATIONS:**

- This product is not sterile, and it should not be used by people with a weakened immune system, e.g. patients suffering from leukemia, whose immune system is weakened by drugs.
- The product should not be used on premature babies and new-born babies in incubators.

**STORAGE:** Store the product at room temperature, in a cold and dry environment, but without an active cooling system. Keep out of reach of children.

**DISPOSAL:** After use dispose of the product according to national regulations.

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The Lentismed logo is a registered trademark of Lentismed d.o.o. TM indicates trademarks of Lentismed d.o.o.